# MONIRI®

## AUTOINFLATION DEVICES

Moniri®-Baby

Moniri®-Kid

Moniri®-Adult

- EN Instructions for Use
- SV Bruksanvisning
- DE Anwendungshinweise
- PT Instruções de uso
- IT Istruzioni per l'uso
- FR Mode d'emploi
- ES Instrucciones de uso

Fig. 1 Moniri®-Baby

Fig. 2 Moniri®-Kid

Fig. 3 Moniri®-Adult

Moniri®-Baby: from 2-8 years

Moniri®-Kid: from 4-8 years

Moniri®-Adult: from 9 years

Fig. 4

Fig. 5


Fig. 6

**Fig. 7** Moniri®-Baby

**Fig. 8** Moniri®-Baby

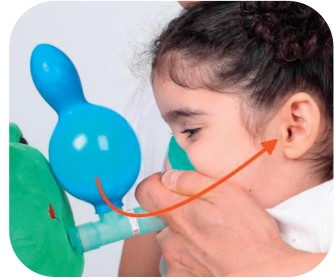**Fig. 9** Moniri®-Baby

**Fig. 10** Moniri®-Kid

**Fig. 11** Moniri®-Adult

# Moniri®

## Instructions for Use

Read the instructions for use before starting the Moniri® treatment.

### Why use Moniri®?

Moniri® can be used to equalize a negative middle ear pressure. This condition is caused by Eustachian tube dysfunction and may follow after an upper respiratory tract infection, an episode of acute otitis media or during a flight. If not resolved, Eustachian tube dysfunction can lead to secretory otitis media (also known as otitis media with effusion or glue ear). If left untreated, this condition may lead to a sensation of heaviness and impaired hearing.

### Who can use Moniri®

Moniri® can be used by children and adults. The treatment of children should always be supervised by an adult. For single patient use only.

### How does the Moniri® method work?

The Moniri® method provides the pressure required to open the Eustachian tube to help equalize the middle ear pressure, ventilate the effusion and achieve symptom relief.

### Product content

Moniri® contains the following components (Fig. 1-3):

- Connecting parts
- Face mask
- Balloons: 5 blue (40 cm H<sub>2</sub>O). 5 red (60 cm H<sub>2</sub>O)
- Teddy Frog with air pump and tube (Moniri®-Baby)
- Moniri® calendar and stickers

Additional balloons are also available separately.

### How often and how long should Moniri® be used?

The treatment period lasts for 4 weeks with 5-10 inflations per session twice a day.

### When should Moniri® not be used?

Moniri® should not be used during an episode of acute otitis media and/or upper respiratory tract infection.

### Warnings and precautions

 The balloons contain LATEX. If the patient is allergic to latex or latex products consult a physician before using this product. If the patient experiences any allergic symptoms, cease treatment and consult a physician.

 This product contains small components and should be kept out of reach from children. The treatment of children should always be supervised by an adult. Only use the balloons provided when using the Moniri® device. Only inflate the balloon until a diameter of approximately 5-10 cm. Larger balloon volumes will lower the balloon pressure and may decrease treatment performance.

## How to assemble Moniri®

Assemble the facemask and mount the balloon on the connecting parts (Fig.4-5). Assemble the connecting part onto the tube in the mouth of the frog (Moniri®-Baby Fig. 6).

## How to use Moniri® (Fig.7-11)

1. In order to show the child how the treatment is performed place the mask on your own nose and inflate the balloon.
2. Place the face mask on the child's face covering both the nose and mouth. The child may assist in holding the face mask in position.
3. Ask the child to inflate the balloon while helping to hold the mask tight to the face in order to create an airtight seal.
4. When required, assist the child by squeezing the frog's belly to pump air into the balloon (Moniri®-Baby).
5. For treatment, inflate the balloon 5-10 times per session twice a day for 4 weeks. Change to a new balloon each week. To reduce the balloon resistance, an adult can inflate the balloons before treatment of the child. Use the Moniri® calendar for treatment monitorization.

## Which balloon shall be used?

**By changing the balloon 2 different inflation pressures can be achieved with Moniri®:**

- Blue balloon (40 cm H<sub>2</sub>O): For standard treatment to achieve clinical effect. If no clinical effect is achieved with the blue balloon, change to the red balloon.
- Red balloon (60 cm H<sub>2</sub>O): For treatment with higher inflation pressure.
- In the event of absence of a clinical effect, as expressed by the child, it is still recommended to complete the 4 week treatment cycle. Contact your physician for evaluation of the treatment.

**A clinical effect can be classified as one of the following:**

- Hearing or sensation of a crackling sound in one or both ears.
- Sensation of moving air or water in the ear.
- Mild transient discomfort in one or both ears (this sensation will decrease with subsequent inflations).
- Improved hearing.

**Non-verbal cues expressed by the child can also indicate an effect. These non-verbal cues can be in the form of but not limited to:**

- Placing a finger into the ear canal or desire to touch the ear.
- Abrupt change in facial expression – startled (wide eyed), smiling or even confused.
- Increased interest to use the device again.

## Maintenance

If required, the face mask and components can be gently rinsed with water or a damp cloth.

Discard used balloons when they have been in use for 1 week of treatment.

Disposal should be made according to local environmental procedures.

## Storage

When not in use, protect the balloons against light. Keep the balloons in the dark bag provided.

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Manufactured by Moniri Medical AB