Moniri®

Nasal Wash

- EN Instructions for Use
- SV Bruksanvisning
- PT Instruções de uso
- ES Instrucciones de uso

2-99

Fig. 1

Fig. 2

Fig. 3

Moniri® Nasal Wash

Instructions for Use

Read the instructions for use before starting.

Why use Moniri® Nasal Wash?

Moniri® Nasal Wash helps children and adults to rinse the nose for prevention and treatment of rhinosinusitis. For single patient use only.

Product content

Moniri® Nasal Wash contains the following components (Fig. 1):

- Connecting parts
- Bottle (300ml)

How to use Moniri® Nasal Wash (Fig.2-3)

1. Dissolve 1 teaspoon of salt into 1 liter of distilled or boiled water.
2. Cool down the saline solution and fill the Moniri® Nasal Wash.
3. Stand slightly bent with the head tilted over a bathroom sink.
4. Press the button and let the saline solution flow from one nostril to the other.
5. Repeat the procedure on the opposite side. Use a tissue to blow out any remaining saline solution.

Warnings and precautions

 If the patient experiences any allergic symptoms, cease treatment and consult a physician. This product contains small components and should be kept out of reach from children. The treatment of children should always be supervised by an adult.

Maintenance

If required, the components can be gently rinsed with water. Disposal should be made according to local environmental procedures.

Storage

When not in use, protect the product against light.

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Moniri Medical AB
Wenzhou Medical Technology Co. Ltd.